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**INSTRUCTION FOR USE****Super Absorbent Dressing Adherent****Description**

Super Absorbent Dressing Adherent is an absorbent wound dressing that helps to maintain a moist environment at the wound surface. It comprises an absorbent core for absorbing and retaining exudates, the absorbent core is enclosed in hydrophobic breathable layer and liquid permeable layer, and the side facing the wound it is covered with a silicone wound contact layer. Super Absorbent Dressing absorbs and retains exudates and MMP's into the absorbent core to reduce the risk for maceration, and encourage wound healing. The gentle silicone allows for easy application, removal and reapplication on the same area, and it can minimize pain to patients and trauma to wound and its surrounding skin.

Dressing material contents: Polypropylene, Polyethylene, Cellulose, Polyacrylate, Silicone.

**Indications**

Super absorbent dressing is intended for use on moderate to heavily exuding wounds, such as: leg ulcers, pressure ulcers, diabetic foot ulcers and surgical wounds.

**Instructions for use**

Super absorbent dressing can be used by health care professionals and lay persons under supervision of health care professionals.

- After wound is cleaned and sterilized, the appropriate size dressing is selected to allow for sufficient space to the wound edge.
- Super absorbent dressing can be used both as a primary dressing or as a secondary dressing. When used as a primary dressing, Super absorbent dressing must be in close contact with the wound bed to ensure wound fluid is absorbed. Secure accordingly with bandage, tapes or compression.
- The frequency of changing dressing depends on the amount of exudates.
- Dressing removal can be facilitated by moistening the dressing with saline solution.

**Precautions and warning**

- The product is single use. Reprocessing devices, in order to reuse them, may seriously damage their integrity and their performance, and Cross-contamination may occur.
- Sterile. Do not use if sterile packaging is damaged or opened prior to use. Do not re-sterilize.
- Do not cut the dressing, the cover must remain intact.
- In the absence of available data supporting the use of this dressing on sensitive population groups such as infants, children, pregnant or nursing women, and in the absence of data to the contrary, on these population groups this dressing should be used with caution and following a clinician's recommendation.

**Contraindications**

- Do not use this product on dry or low exuding wounds, eyes, mucous membranes or in wound cavities because the dressing swells during absorption.

- Do not use on third-degree burns.
- Super absorbent dressing should not be used on persons with known allergy or hypersensitivity to any of its components.

**Storage**

Keep in a clean and dry place in specified temperature range and away from sunlight;

**Disposal**

Dispose should be handled according to local environmental procedures.

**Other information**

If any serious incident has occurred in relation to the use of Super Absorbent Dressing, it should be reported to the manufacture and competent authority of the Member States.

**The explanation of the Graphics, symbols and abbreviations for use in the label**

Symbols	Explanations	Symbols	Explanations
	Manufacturer		Authorized representative in the European Community
	Date of manufacture		Use-by date
	Catalogue number		Sterilized using ethylene oxide
	Batch code		Do not re-sterilize
	Do not re-use		Caution
	Temperature limit		Medical Device
	Consult instructions for use		Do not use if package is damaged
	Unique device identifier		Keep away from sunlight
	Keep dry		CE mark and identification number of Notified Body
	Single sterile barrier system		This way up
	Importer		

**Shelf life:** 3 years from date of manufacture

**Manufacturer:** Huizhou Foryou Medical Devices Co., Ltd

**Address:** North Shangxia Rd. Dongjiang Hi-tech Industry Park, 516005, Huizhou,

PEOPLE'S REPUBLIC OF CHINA

**Tel:** +86-752-5302185 **Fax:** +86-752-5302020

**Authorized representative in the European Community:** Shanghai International Holding Corp. GmbH (Europe).

**Address:** Eiffestrasse 80, 20537 Hamburg, Germany

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**Netherlands** B.V.
**Germany** GmbH